


Loading Instructions

Akreos® AO and Akreos® AO MICS preloaded with the Bausch + Lomb SimplifEYE™ delivery system

BAUSCH + LOMB™
SimplifEYE™
delivery system




1

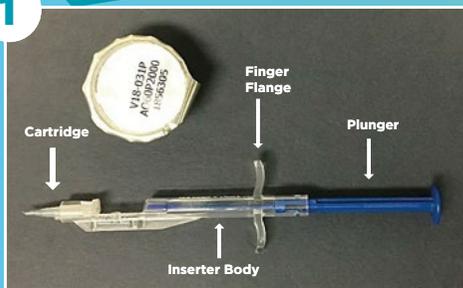


Open the box and remove the inserter and IOL pouch. Open the peel pouch to take the IOL vial out. Peel the tyvek lid and remove the inserter from the tray.

2

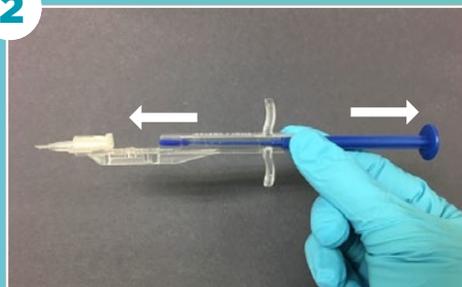


Ensure that the plunger is fully retracted by pulling it to the back and that the cartridge is fully forward by pushing it forward. The inserter is now ready for assembly with the lens shuttle.

3

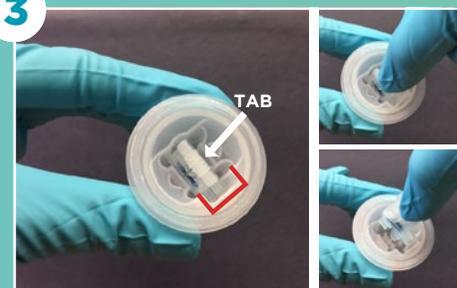

Remove the lid from the vial by peeling it away from you. Ensure that the Uchannel (highlighted in red) in the vial is facing towards you. Using the thumb and index finger to remove the lens shuttle from the vial.

4

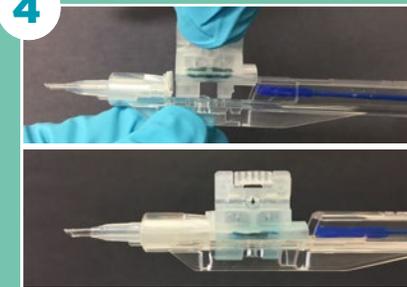


Grasp the inserter body with one hand and the shuttle with the other hand. Ensure that the four legs of the shuttle are centered to the inserter body side walls and insert the shuttle straight down until you hear an audible click. Visually confirm that the shuttle is sitting flush and that the top of the shuttle is horizontal with all four legs inside the side walls of the inserter body.

5




Hold down the tab on the top of the shuttle and apply viscoelastic* into the viscoport of the cartridge. Visually verify that the viscoelastic has traveled up to the mark as shown in the image.

6

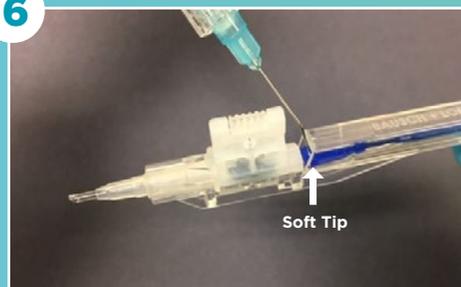


Add a drop of viscoelastic* to the soft tip for easy entry into the shuttle.

7

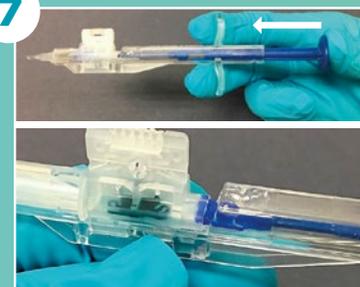

While holding the inserter body with one hand, gently advance the plunger. Visually confirm that the soft tip enters the shuttle without deforming.

8

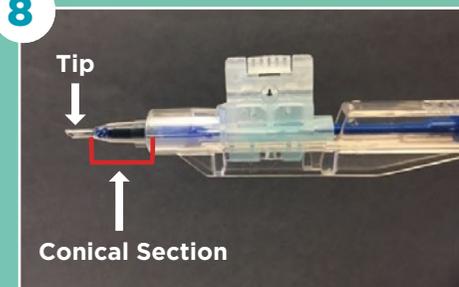

Continue to advance the plunger until the lens is in the conical section of the cartridge tip, as shown in the image. To deliver the lens, insert the cartridge tip into the incision with the tip bevel facing down. Slowly advance the plunger until the lens is fully released into the eye.

*Amvisc®, Amvisc® Plus and OcuCoat®

Loading Instructions

Akreos® AO and Akreos® AO MICS preloaded with the Bausch + Lomb SimplifEYE™ delivery system

BAUSCH + LOMB
SimplifEYE
delivery system



Correct




Correct orientation of shuttle

Correct



Correct

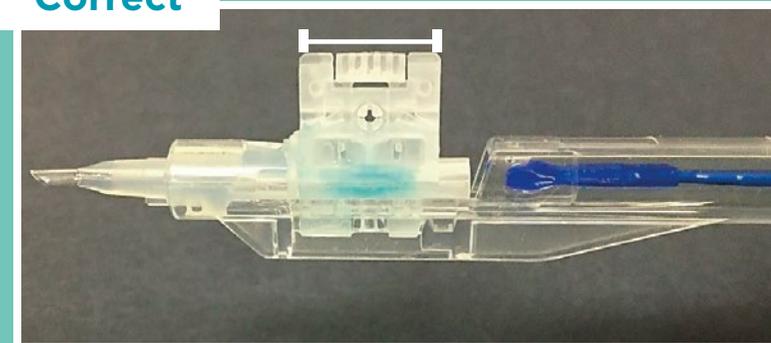

Incorrect




Incorrect orientation of shuttle

Incorrect



WARNING:
Do not roll or tilt the shuttle while attaching to the inserter.

Incorrect

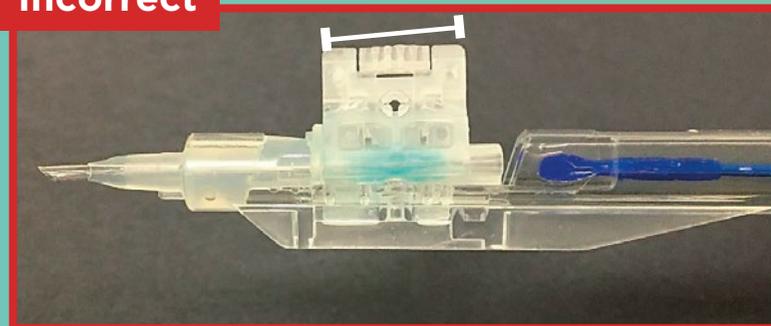

WARNING:
The shuttle on the bottom is not the correct way to assemble the shuttle because it is not horizontal to the inserter the body and could cause damage to the lens when plunger is advanced.
To correct this:
Push the shuttle down until it is horizontal.